


# MOBIUS BRACE SIZING GUIDE

1. Print measuring guide at 100% Scale in landscape format. Do not resize or scale to fit page.
2. Cut out measuring guide on black lines and use adhesive tape to attach pieces end to end.
3. Follow directions on guide to determine correct size.

In some cases, a Mobius Fit Kit may be necessary to obtain a perfect fit.

For any further questions, please visit our website at [www.MobiusBrace.com](http://www.MobiusBrace.com) or call us at 208 928-7850.

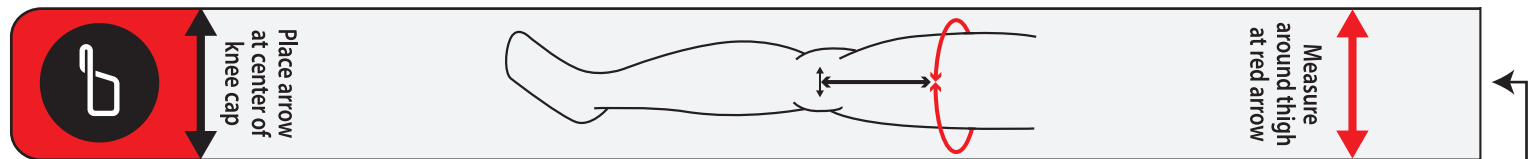

CUT OUT AND ATTACH ENDS. DO NOT OVERLAP

## HOW TO DETERMINE THE CORRECT SIZE OF MOBIUS BRACE:

From the center point of the kneecap, measure 6 inches (15.25cm) vertically, or up the thigh, then measure the circumference or diameter of the thigh. If the measurement falls between two sizes, select the larger size. The optional Pad Fit Kit may be necessary to get a precise fit.

CUT OUT AND ATTACH ENDS. DO NOT OVERLAP

<b>SMALL</b> 16"-18" 40.6cm-45.7cm	<b>MEDIUM</b> 18"-20" 45.7cm-50.8cm	<b>LARGE</b> 20"-22" 50.8cm-55.9cm	<b>mobius</b> ®
--	---	--	-----------------

